

DISTRIBUTED BY:



www.BrasselerUSA.com

One Brasseler Boulevard, Savannah, GA 31419

Nose Pad Installation.

- Remove the existing blue nosepad using a small screwdriver. NOTE: an optical screwdriver was provided with your BrasselerVE Loupes.
- Attach the new nosepad using the screw provided. Ensure the nosepad is facing the correct direction by aligning the two small nubs on either side of the nose pad's screw hole with the corresponding divots on the inside of the frame.



Temple Arm Installation-

- Using a small screwdriver, remove the existing blue/silver temple arms by unscrewing the screws on the under-side of each temple arm.
- Attach new temple arms using screws provided. NOTE: Because the BrasselerVE frame features spring-loaded hinges for added flexibility, the screw holes in the new temple arms will not line up with the frame's screw holes. This is normal.

To better align the holes for easy installation, insert the new temple arms to the frame in a closed position (as shown in picture) Insert the screws at an angle. As you turn the screws, they will straighten and guide themselves into position.



Side Shield Installation-

- 1. Remove the existing side shields by pulling them straight down, perpendicular the temple arms.
- To attach the new side shields to the temple arms, align the two pegs on each side shield with the two holes on the under-side of each temple arm and snap into place.

NOTE: Retain your original accessories. If any exchanges or returns are needed during the 30 day trial period, the loupes must be returned with all original accessories intact to avoid restocking fees.

